



Vibrating

6 training courses

App operated

Additional cover


Made from silicone

Kegel trainer

PRODUCT MANUAL

Charging

- For maximum pleasure, charge before every use.
- Connect the included USB cable and insert the USB end into the wall adaptor or computer USB port (Computer must be turned on).
- The slow intermittent glow indicates charging in progress.
- A full charge takes up to 90 minutes for 150 minutes of play.
- Do not use the device while charging.

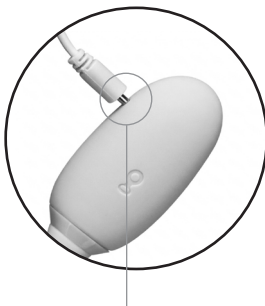

USB charging port

How To Use

- Hold the egg's button to turn on/off.
- The device vibrates when squeezed: the harder the squeeze, the more intense the vibration.



Power button

App operation

Download and installation

- Download the Magic Kegel app from Google Play or App Store (the app is free).

QR Code



- Open the app - it will request access to the following: Bluetooth, Location Information, Storage permissions, and Microphone.
- Please allow all access in your phone's Settings.
- Accept the User Agreement. Log in to an existing account or register a new one.
- Hold the egg's button to turn the toy on/off.
- Tap the "Connect the Kegel Ball" button to link the toy to your phone.
- The toy must be turned on to establish a connection to the app.
- Follow the instructions on the screen to link the device.
- Tap the play ► button, to start the workout.
- Follow the instructions on the screen to enjoy a stimulating massage during each exercise.
- Choose a workout from 6 suggested exercise programs: Entry level, Planning a baby, Muscle recovery, Self care, New mom , Enhanced intimacy.
- Tap the ✓ button to customize the vibration intensity of the toy.

Cleaning and Care

- Use a toy cleaner/mild soap with water to clean the product.
- Pat with a towel and allow it to air-dry completely.
- Use with the water-based lube only, no silicone lubricants.
- Please avoid contact with silicone or silicone-containing materials as it may damage the product.

Warning

- Do not use on inflamed and injured areas.
- Discontinue use if feeling pain or discomfort.
- Adult novelty only. Use for medical purposes or for a use that has an adverse effect on any function of the body is prohibited.



edenfantasys
THE NEW INTIMACY

www.edenfantasys.com

Customer Service
10 AM - 7 PM EST
Phone: 1 (770) 604 - 1352